

THE ACUVUE® ULTRA COMFORT SERIES.™


AN INNOVATION IN COMFORT, VISION AND HEALTH.

ULTRA smooth & flexible

- Deliver consistently clear vision by conforming easily to the shape of your eye.
- Feel unbelievably soft due to the smooth surface design and material flexibility that's greater than other high-oxygen contact lenses.

ULTRA breathable

- Help keep your eyes fresh and white, so they look as good as they feel.
- Allow nearly 100 percent of available oxygen through.

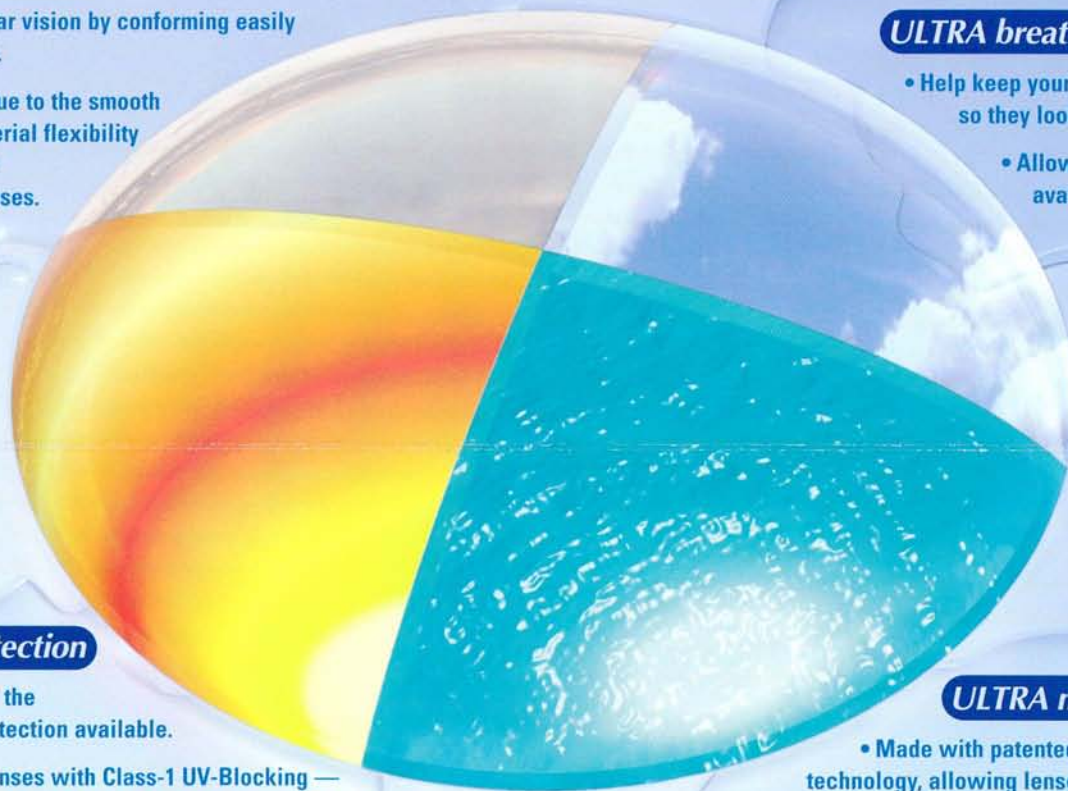

ULTRAViolet protection

- Help protect eyes with the highest level of UV protection available.
- Are the only contact lenses with Class-1 UV-Blocking — they block more than 90 percent of UV-A and 99 percent of UV-B radiation.

ULTRA moist

- Made with patented HYDRACLEAR™ technology, allowing lenses to stay comfortable all day.
- Provide high moisture retention to create a unique, silky, soft feel on your eyes.

Contact lenses currently in the
ULTRA COMFORT SERIES™:

ACUVUE®
ADVANCE™
with HYDRACLEAR™

ACUVUE®
ADVANCE™
FOR ASTIGMATISM

ACUVUE®
OASYS™
with HYDRACLEAR® Plus

Important information for contact lens wearers: ACUVUE® Brand Contact Lenses are available by prescription only for vision correction. An eye care professional will determine whether contact lenses are right for you. Although rare, serious eye problems can develop. To help avoid these problems, follow the wear and replacement schedule and the lens care instructions provided by your eye doctor. Do not wear lenses if you have an eye infection, or experience eye discomfort, excessive tearing, vision changes, redness or other eye problems. If one of these conditions occurs, contact your eye doctor immediately. For more information on proper wear, care and safety, call 1-800-843-2020 or visit acuvue.com.

WARNING: UV-absorbing contact lenses are NOT substitutes for protective UV-absorbing eyewear such as UV-absorbing goggles or sunglasses because they do not completely cover the eye and surrounding area. You should continue to use UV-absorbing eyewear as directed. **NOTE:** Long-term exposure to UV radiation is one of the risk factors associated with cataracts. Exposure is based on a number of factors, such as environmental conditions (altitude, geography, and cloud cover) and personal factors (extent and nature of outdoor activities). UV-blocking contact lenses help provide protection against harmful UV radiation. However, clinical studies have not been done to demonstrate that wearing UV-blocking contact lenses reduces the risk of developing cataracts and other eye disorders. Consult your Eye Care Practitioner for more information.

HYDRACLEAR™ is our brand name for our proprietary technology that brings together a remarkable moisture-rich wetting agent with high-performance base materials to form clear, wearable lenses, resulting in a unique, silky, soft feel. ACUVUE® 2 **COLORS™**, ACUVUE® ADVANCE™, ACUVUE® OASYS™, HYDRACLEAR™, ACUVUE® COMFORT SERIES™ and FOR THE LIFE OF YOUR EYES™ are trademarks of Johnson & Johnson Vision Care, Inc. © Johnson & Johnson Vision Care, Inc. 2006.

ADVANCED
EYE CARE & SURGERY

501 E. Palm Valley Blvd.
Round Rock, Texas 78664
(512) 248-2424
www.eyecare-surgery.com

THE SATISFACTION GUARANTEE

If for any reason you are not satisfied with your contact lenses, return them within 90 days and receive your money back. Other terms and restrictions apply. Visit acuvue.com or ask your eye doctor for complete details.

With the ULTRA COMFORT SERIES, your satisfaction is guaranteed.

ACUVUE® BRAND PROMISE

We promise to honor your trust in ACUVUE® by providing products that deliver the highest levels of vision care and comfort.

- Please read your Patient Instruction Guide and talk to your eye doctor for complete wear and care information.
- **Before handling lenses:** Make sure your hands are clean and dry; avoid soaps with moisturizers.
 - **To apply lenses:** Firmly hold your upper lid. Either look up and place the lens below the center of your eye or look down toward where you've placed the lens to center it before gently releasing your eyelid. Do not look directly at your finger as it approaches your eye.
 - **To remove lenses:** Tilt your chin down, look up slightly. Gently nudge the lens down with your index finger; apply gentle pressure against your eye with your thumb and index finger, and then softly squeeze the lens to remove it.
 - **Cleaning lenses:** Gently rub and rinse your lenses before storing, even if using a "no rub" solution. It helps refresh the surfaces of the lenses.
 - **Replacing lenses:** Change your lenses as often as recommended by your doctor. Lenses that feel fine may have changes in the lens surface that can irritate your eye.

Wear and care tips for your ACUVUE® ULTRA COMFORT SERIES™ contact lenses.

The products in the ULTRA COMFORT SERIES™ provide outstanding comfort, vision and health:

AGAIN.

ACUVUE® ULTRA COMFORT SERIES™ lenses offer exceptional moisture and breathability, smooth, flexible lens design and unmatched ultraviolet protection,** making them the most comfortable high-oxygen contact lenses available. And because the ULTRA COMFORT SERIES™ is from the makers of ACUVUE® – the #1 doctor-prescribed brand – you can feel confident that the lenses will provide you crisp, clear, comfortable, healthy vision all day long.

ACUVUE® ADVANCE™ Brand with HYDRACLEAR™

- Long-lasting, all-day comfort.
- The #1 doctor-prescribed high-oxygen (silicone hydrogel) contact lenses.
- Available if you're nearsighted or farsighted – great for new wearers!

ACUVUE® ADVANCE™ Brand for ASTIGMATISM

- Breakthrough Accelerated Stabilization Design and HYDRACLEAR™ technology provide crisp, stable vision, no matter what you're doing.
- The first high-oxygen lens in the U.S. specifically designed for astigmatism.

NEW! ACUVUE® OASYS™ Brand with HYDRACLEAR™ Plus

- Exceptional comfort and crisp, clear vision, even in situations where eyes can feel tired and dry.
- Significantly reduce the need for rewetting drops.
- It's the "best contact lens ever tried," according to most patients who wore it in a recent clinical study.

