

AS *easy*  
as 1-2-3.

Just follow these 3 simple steps:\*\*



### 1. RINSE

Place lenses in basket holder and rinse with Clear Care® for 5 seconds.



### 2. FILL

Fill case to "fill line" with Clear Care.



### 3. SOAK

Store lenses in case 6 hours or overnight.  
**DO NOT REMOVE BEFORE 6 HOURS.**

## IMPORTANT TIPS:

Making the switch from a multi-purpose solution to Clear Care® is easy. Just remember a few important tips.



Never use with a flat lens case. Clear Care must be used with the special lens case provided.



Because the solution needs time to neutralize, never remove lenses from case until at least 6 hours later.



Never rinse lenses with Clear Care prior to inserting lenses into your eyes. If you want to rinse lenses, use a sterile saline solution.



Never squirt Clear Care directly into your eyes.

Keep your soft contact lenses **clean** and keep them **comfortable** from the start till the end of the day with Clear Care — clinically proven

*#1 in comfort.*<sup>2,3</sup>

To learn more,  
visit [clearcaresolution.com](http://clearcaresolution.com)



**CIBA VISION.**  
Shared Passion for Healthy Vision and Better Life

\*When used as directed. \*\*Practitioner instructions may differ. See package insert for complete instructions.

References: 1. CIBA VISION data on file, 2008. On average vs. OPTI-FREE® RepleniSH® 2. CIBA VISION data on file, 2006. According to subjective ratings given by silicone hydrogel lens wearers in a clinical study comparing Clear Care to OPTI-FREE RepleniSH, OPTI-FREE® EXPRESS®, COMPLETE® MoisturePLUS™ and ReNu MultiPlus® as a group. 3. Dillehay SM, McCarter HE, et al. A comparison of multi-purpose care systems. *CL Spectrum*. 2002; April: 30-36.

OPTI-FREE, RepleniSH and EXPRESS are registered trademarks of Alcon, Inc. COMPLETE is a registered trademark and MoisturePLUS is a trademark of Advanced Medical Optics, Inc. ReNu MultiPlus is a registered trademark of Bausch & Lomb, Inc.

© 2008 CIBA VISION Corporation 2008-12-1223 100018

The end of the day  
never felt  
*so good*



The powerful clean  
WITH A *gentle touch*

## Add an extra hour of contact lens comfort to your day with Clear Care.<sup>®1</sup>

In a recent study, people using Clear Care enjoyed, on average, **one extra hour of comfortable wearing time** compared to OPTI-FREE<sup>®</sup> RepleniSH<sup>®1</sup>. In fact, Clear Care is clinically proven **#1 in comfort**.<sup>2,3</sup>

What will you do with your

*extra hour?*

*Gentle,*  
for a difference you can *feel*

### No MPS preservatives — gentle on the eyes

- After a six-hour soak in the unique Clear Care lens case, it leaves behind a gentle saline solution.
- Clear Care deep cleans contact lenses overnight with no added MPS preservatives to irritate eyes.




*Powerful,*  
for a difference you can *see*



Exceptional cleaning action of hydrogen peroxide cleans and disinfects lenses like no multi-purpose solution can.


**With its bubbling action, you can actually see it working!**

### Powerful triple-action

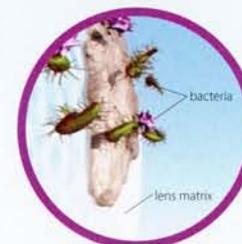

#### 1. DISINFECTS

Hydrogen peroxide penetrates the lens and kills germs and bacteria for strong disinfection and deep cleaning.

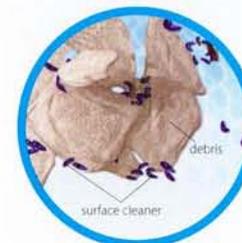

#### 2. CLEANS

A built-in surface cleaner breaks down and lifts away debris and protein deposits.



#### 3. REMOVES PROTEIN

The unique formula of Clear Care bubbles to enhance surface protein removal.

